



# Calligator

Special rinse aid for use in DISCHER bedpan washer-disinfectors  
Liquid concentrate



<b>Main field of application:</b>	Against lime precipitation in steam generators of bedpan washer-disinfectors
<b>Properties:</b>	<p>Calligator is a mildly acidic rinse aid with special components for effectively complexing water hardness. Due to the good lime bonding ability and the excellent wetting properties the items to be washed are rinsed stain-free and without lime deposits.</p> <p>When used as intended no lime deposits develop in steam generators or in pipes.</p> <p>Calligator can be used with all degrees of water hardness.</p>
<b>Application and dosage:</b>	<p>Calligator is used in DISCHER bedpan washer-disinfectors. Dosage: 0.5 – 1.5 ml/l, depending on water hardness via integrated dosing devices. Dosing systems and steam generators must be acid-compatible. This has been tested and approved for DISCHER bedpan washer-disinfectors.</p> <p>Do not mix with other products.</p> <p>Rinse out dosing system including suction hose with water before changing product.</p> <p>Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated processes.</p> <p>Please observe the reprocessing information of the medical device manufacturers according to the requirements of the DIN EN ISO 17664.</p> <p>The instructions given by the manufacturer of the washer-disinfector are to be observed.</p> <p>For professional use only.</p>
<b>Technical data:</b>	<p>Density (20 °C): approx. 1.1 g/cm<sup>3</sup> pH-range (determined in deionised water, 20 °C) 0.5 – 1.5 ml/l: 3.4 – 3.1 Viscosity (concentrate, 20 °C): &lt; 50 mPas</p>
<b>Ingredients:</b>	Ingredients according to Regulation (EC) No 648/2004 on detergents: < 5 % polycarboxylates, also: preservatives (methylchloroisothiazolinone /methylisothiazolinone)
<b>CE-mark:</b> 	<p>Calligator complies with European guidelines for medical devices.</p> <p>If a serious incident occurs with the product, report it to the manufacturer and the relevant national authority.</p>
<b>Storage information:</b>	Store in a frost-free place. Always store at a temperature between 0 °C and 30 °C. Usable for 3 years when stored as recommended. For expiry date refer to the stamp mark on the label behind the hourglass symbol 
<b>Hazard and precautionary statements:</b>	<p>For safety information see Safety Data Sheets.</p> <p>Dispose only when container is empty and closed. For disposal of product residues, refer to the Safety Data Sheet.</p>

# Calligator

Special rinse aid for use in DISCHER bedpan washer-disinfectors  
Liquid concentrate



**Distribution:**

Distributed by: DISCHER Technik GmbH, Fuhr 4-6, 42781 Haan-Gruiten, Germany,  
Tel: +49 (0) 2104 2336-0, [info@discher.de](mailto:info@discher.de), [www.discher.de](http://www.discher.de)

**Manufacturing:**



Manufactured by: Chemische Fabrik Dr. Weigert GmbH & Co. KG, Mühlenhagen  
85, 20539 Hamburg, Germany, Tel.: +49 (0)40 7 89 60-0

*With the above information, which is appropriate to the current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.*

Date of issue: 06/2020